



Drying Device

for dental ceramics



Please read this manual carefully before using the device and keep it at a save place for a later use.

Important Safety Instructions

- Disconnect from power when the device or parts of it are visibility damaged.
- Disconnect the device from the power supply prior to cleaning.
- Do not clean under running water or immerse in water.
- The device is intended solely for the drying of ceramic structures.

The manufacturer does not accept any damages resulting from unintended use or inappropriate handling. Modifications of the product are always possible without notifying it in advance.

Description Of The Drying Process

An entire drying of the colored material prior to the sintering is essential for both, the result of the coloring and the protection of the sintering furnace.

Recommended drying time:

- Single crowns: > 30 minutes
- Bridges from 2 to 6 pieces: approx. 90 minutes
- Bridges from 7 to 14 pieces: approx. 120 minutes

Time Display

shows the remaining time during timer mode

Power Levels

Level I: approx. 270W
Level II: approx. 520W

Starting / Stopping

Preset Time

in 30 minutes steps



Operation


1. Switch on the device
2. Adjust time using + or -
3. Device stops automatically after given time

Hint

Press the power-button in order to switch off the device before the preset time. It will shut down after a delay of approx. 5 seconds.

Important Advice

The lid has to be brought into the upper position prior to operation.
Allow unit to cool down after use.
Not designed for continuous operation.

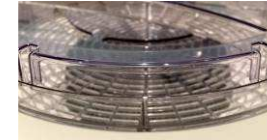

Technical Data

Voltage	230 V AC / 50Hz
Power	≤ 520 W
Temperature	≤ 80 °C
Dimensions (D/H)	320 x 200 mm
Weight	1.8 kg

Made in EU



Thermo-Star GmbH
Krantzstr. 7 / Geb. 37
52070 Aachen, Germany

Phone +49 241 60845-0
Fax +49 241 60845-100
info@thermo-star.de
www.thermo-star.de

